

QUICKREF: Lightwriter SL5 and SL55

MEM

- The MEM-key has many uses:

MEM	this level is for recalling memories.
MEM-MEM	this level is for saving memories.
MEM-MEM-MEM	this level is for executing functions.
- There are 36 memories

26 of the type MEM A, MEM B, MEM C, etc; (each letter-key can store the first 127 characters of a message; this includes spaces).
10 of the type MEM 1, MEM 2, MEM 3, etc; (each number-key can store the first 255 characters of a message; this includes spaces). The machine will beep when 230 characters are entered, as a warning.



STORE A MEMORY

- Press: ON/C, type your message, then MEM MEM then the desired letter or number.
- The screen-message is Replace ?? (even if that memory-location is empty); press Y for 'yes', any other key means 'do not change the old memory'.

EXAMPLE:	ON/C then type LIGHTWRITER then
MEM MEM L Y	

RECALL A MEMORY

- Press: MEM then [chosen key]; the speed of recall is controlled by the setup 'Speed'.

REPLACE A MEMORY

- As for 'store'. Pressing Y means the old memory will immediately be replaced by the new.

ERASE ONE

- Press: ON/C MEM MEM then [chosen key] then Y.

ERASE ALL for a new user

- Press: MEM MEM then hold ? for 10 seconds, until 'Erasing' is displayed; wait until Erasing disappears.

SETUPS

- To enter the setup system Press: MEM REPLAY; the display will be 'Text' (the first setup).
- To browse the setup list Press: SPACE for next; BACK-SPACE for previous.
- To change a setup Press: + for next value; EXPERIMENT!
- To leave the setup-system Press: ON/C.

THE SETUPS ARE

Text	Select UPPER- or lower-case.
Click	Select key-click (for auditory feedback).
Tremor	Select anti-ttrreemmmor (for settings other than 0, hold the key down).
Speed	Select memory-speed and replay-speed.
Auto-off mins	Select turn-off time (the Lightwriter turns off automatically if unused).
Keys	Select symbols/accents on the two keys to the right of 'L'
Br=British, Am=American, Da=Danish and Norwegian, Sw=Swedish and Finnish, Ge=German, Fr=French, Sp=Spanish and Portuguese, Ja=Japanese.	

KEYS



SHIFT-ONCE

- Press: SHIFT once to shift the following character only.

EXAMPLE	SHIFT PERIOD for COMMA
EXAMPLE	SHIFT 5 for %

SHIFT-LOCK

- Press SHIFT twice to shift all following characters; press a 3rd time to unlock. Note: if the setup is 'lower-case', the shift-key produces UPPER-CASE letters.

CASE

- You can type in UPPER or lower case, using the 'Text' setup.

REPLAY

- This key will repeat the message that you have typed; the Replay-speed is controlled by the setup 'Speed'.
- You can type indefinitely, but you can only recall the first 255 characters of a message.

CHARGER

- A dot at left of display = Low battery! Give it an overnight charge, USING THE CHARGER SUPPLIED.
- If you plan to travel: the existing charger will work (with a suitable adaptor), provided the supply is 220 - 250 volts (Europe, most of Australia, Now Zealand, etc).
- The charger output must be 9-10 VDC, minimum rating of 800mA, output-plug: 2.1 mm x 5.5 mm x 12 mm, positive-tip, 12V. A car charging-lead is available; this charges the equipment from a car cigarette-lighter socket.
- Capacity: 10 hours continuous use. If the battery is low, use the Lightwriter with the charger plugged in.

USE ALL DAY, CHARGE EACH NIGHT

You should also have Literature Part No: INFO LIT542CH 24 October 2007 (DynaVox)

This is Literature Part No: QUICKREF LIT541CG 24 October 2007 (DynaVox)

Designed and manufactured in the UK by:

TOBY CHURCHILL Limited
 Norman Way Industrial Estate,
 OVER,
 Cambridge, CB4 5QE,
 United Kingdom
 Tel: (+44) 1223 576117
www.toby-churchill.com

Distributed in USA and Canada by:

DynaVox Technologies
 2100 Wharton Street Suite 400
 Pittsburgh, PA 15203
 1-866-396-2869
<http://www.dynavoxtech.com>



Technology for human expression.

INFO: Lightwriter SL5, SL55

SOFTWARE VERSION

- Press: MEM MEM then ?

QWERTY TO ABCDE

- Return to your supplier.

CARE OF THE EQUIPMENT

- Food, drink, and mechanical abuse will do the equipment no good at all! Do not drop it; arrange a suitable mounting-system, if required.

CE STATEMENT

- An indirect-field static-discharge (eg lightning) may cause the device to 'lock up' (if it is in use at the time). In this circumstance, the user should press OFF then ON.

AIRCRAFT

- If you are asked by the cabin-crew not to use electrical equipment during parts of the flight such as take-off or landing, you should also refrain from using the Lightwriter during these times. Use the alphabet-card supplied.
- Carry the equipment as hand-luggage and not in a suitcase or bag; show airport-security staff how it works; this may make the security-check faster. The equipment will not be affected by the security X-ray machines.

STERILISATION

- If it is required to sterilise the equipment, the outer surfaces may be wiped with a cold-sterilising agent.
- The equipment is splashproof (not waterproof); do not allow sterilising agent or cleaning fluid to get inside.
- The recommended technique is to hold equipment on end with the charger socket lowest and to clean the equipment in this position, reducing the risk of fluid entering the equipment.
- Do not use strong sterilising agents such as surgical spirit. Do not immerse in sterilising fluid.
- Do not expose to elevated temperatures. Do not autoclave.
- If a potential user is suffering from a virulent condition from which the equipment cannot be adequately sterilised using the above techniques, take care to avoid the risk of infecting future users or other people handling the equipment; you may wish to use an alphabet or picture board, which can be disposed of after use.

RETURNING THE EQUIPMENT

- If you need to return the equipment to your supplier, here is what to do:
 - Pack the machine and its charger with plenty of protection, preferably in the original box.
 - Ensure the ON button cannot be pressed in transit.
 - Tape up the parcel securely.
 - Return all the equipment (including charger) to your supplier.
 - Address it clearly; and put the sender's name and address on the back.
 - Enclose a note to say who it is from, and why, and supply the RETURN ADDRESS.
- Ring, or send a letter separately, quoting:
 - Your name, address and phone-number.
 - The serial number (on the back).
 - The account number (if known).
 - The date of dispatch.
 - The faults (or reason for return).
 - The company-name of courier.

RESET

- 'Reset' on SL5 is Off/On. If this does not cure the problem and the Lightwriter is accepting charge, it needs to be returned.

SERVICING

- The Lightwriter is extremely reliable if treated properly.
- By far the most frequent cause of damage is mechanical abuse (dropping it, coffee-spillage, etc).
- The equipment does not need regular maintenance, and is designed to give many years trouble-free use.

TROUBLE-SHOOTING

- Most problems are to do with battery-charging:
 - | Is the mains-socket OK?
 - | Is the charger-light lit?
 - | Try wiggling the wire
 - | Are you using our charger, and is it turned on?
 - | If not, the charger may be dead.
 - | It may have broken inside the insulation, if it has been pulled violently.

ACCESSORIES

KEYGUARD

- If the green keyguard spacers do not provide a deep enough keyguard, we supply an alternative keyguard with deep recesses. This can be fitted by the user.

KEYBOARD GLOVE

- A clear plastic sheet which fits over the keyboard, to reduce the risk of splash-damage.

RAINCOAT

- Same but covers entire machine, and provides even more splash-protection. Specify serial-number.

CARRYING

Neck strap:	Clips onto the keyboard's carry-handle; supplied on request.
Carry bag:	Soft, padded material
Protective case:	Executive-style suitcase.
Mounting:	MidiMount wheelchair and floor-stand system.

You should also have Literature Part No: QUICKREF LIT541CG 24 October 2007 (DynaVox)

This is Literature Part No: INFO LIT542CH 24 October 2007 (DynaVox)

Designed and manufactured in the UK by:

TOBY CHURCHILL Limited
 Norman Way Industrial Estate,
 OVER,
 Cambridge, CB24 5QE.
 United Kingdom
 Tel: (+44) 1223 576117
www.toby-churchill.com

Distributed in USA and Canada by:

[DynaVox Technologies](http://www.dynavoxtech.com)
 2100 Wharton Street Suite 400
 Pittsburgh, PA 15203
 1-866-396-2869
<http://www.dynavoxtech.com>



Technology for human expression.