


## Lightwriter Scan Module

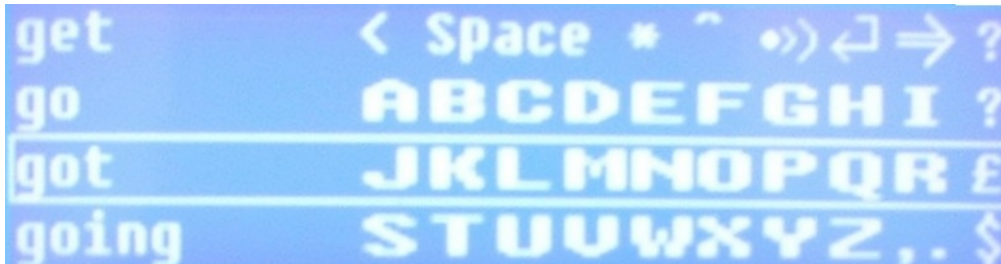

**Configure the Lightwriter.** Press **MEM** then **Replay** then **P** repeatedly to go to the setup **Port 1** and **+** repeatedly to change the option to **Port 1: Scanner**

**Insert the Scan-Module's connector into the right-hand side of the Lightwriter.**

**Insert a click-switch in Switch 1.** Insert a second click-switch in **Switch 2**, if desired.

**Click either switch to turn on.** The scan screen will 'appear'; a scan will start automatically.

**Full details of the scan-system in Section 6 of the Manual.**

- The scan-module can also be used by keyboard-users:
  - The message can be duplicated on the scan-screen, in a choice of 3 modes:
    - | 4 lines of **small** text
    - | 2 lines of **medium** text
    - | 1 line of **large** text
  - Or, 8 predictions can be displayed on the scan-screen; keys **1, 2, 3 ... 8** select the corresponding prediction.
- The setup for any of these is **Mem Replay Shift Period** then **Backspace** repeatedly to **Kbd Screen**.
- Press **+** repeatedly to browse the options. **On/C** to select.

### USE ALL DAY, CHARGE EACH NIGHT

Literature Part No: **LIT520BK 19 November 2007 (DynaVox)**

Designed and manufactured in the UK by:

TOBY CHURCHILL Limited  
Norman Way Industrial Estate,  
OVER,  
Cambridge, CB24 5QE.  
United Kingdom  
Tel: (+44) 1223 576117  
[www.toby-churchill.com](http://www.toby-churchill.com)

Distributed in USA and Canada by:

DynaVox Technologies  
2100 Wharton Street Suite 400  
Pittsburgh, PA 15203  
1-866-396-2869  
<http://www.dynavoxtech.com>



Technology for human expression.