

# Mini Voice Amp

## Operating Instructions

The Mini Voice Amp is portable, lightweight and convenient to use, allowing you to speak out whenever you want. It features a discreetly designed lightweight head-worn microphone. The volume adjustment gives you control over your voice, making it perfect for use in a number of personal or professional settings.

Please read these instructions carefully before use and then store them safely for future reference.

### Instructions for use

1. Open the packaging and check all the contents are present
2. Insert the batteries into the two battery housing areas in the back of the unit, making sure they are sitting the correct way round for polarity (+ and -)
3. Make sure the volume dial is turned towards 'OFF' until it clicks
4. Plug the power adaptor into the side of the unit (DC-9V/250mA) and then into the wall. The red light (ON) will come on to show that power is getting to the unit and that it is in 'charge mode'
5. When the light turns to green the batteries are fully charged. Remove the power adaptor and the unit is ready for use
6. Fit the head-worn microphone around the back of your head so that the two loops fit comfortably over your ears and the microphone sits close to your mouth. Plug into the MIC IN socket on the unit.
7. Click the volume dial 'ON'. The red 'PW-ON' light should come on to show that the unit is now switched on
8. Speak into the microphone and adjust the volume setting as required
9. Remember to switch off the power when not in use to ensure optimum battery life

### Specifications

Frequency response	50Hz-12KHz
Max output power	6W
Output impedance	4 Ω
Operating voltage	6x 1.2V Ni-Mh rechargeable or 6x 1.5V Alkaline batteries
Operating temperature	-10-45°C
Charge time	6-8 hours
Operating time	4-6 hours continued use (new/ fully charged batteries)

### Contents of box

<ul style="list-style-type: none"> <li>• mini voice amplifier</li> <li>• adjustable waist strap</li> <li>• belt clip</li> <li>• head-worn microphone*</li> <li>• 6x rechargeable AAA batteries</li> <li>• operating instructions leaflet</li> <li>• power adaptor</li> </ul> <p>*Microphone description and specification will depend on your purchase option. Assessment kit contains: 3x Headworn Microphones, Throat Microphone and Lapel Clip Microphone.</p>
---

### Important notes

- Make sure the batteries are fully charged before their first use
- Do not leave the batteries in the unit when not in use for a prolonged period of time
- When the power light starts to dim, the battery power is low. Change or recharge the batteries as soon as possible
- For best results, make sure the source of sound is close to the microphone
- Keep the microphone away from the speaker to avoid any feedback on the amplifier
- Keep the unit upright and use the waist strap provided when in use
- The unit will operate using the mains adaptor without batteries. For your safety, we do not recommend that this method is used unless in the case of an emergency

USERMVA2EN

# Mini Voice Amp

## Operating Instructions

The Mini Voice Amp is portable, lightweight and convenient to use, allowing you to speak out whenever you want. It features a discreetly designed lightweight head-worn microphone. The volume adjustment gives you control over your voice, making it perfect for use in a number of personal or professional settings.

Please read these instructions carefully before use and then store them safely for future reference.

### Instructions for use

1. Open the packaging and check all the contents are present
2. Insert the batteries into the two battery housing areas in the back of the unit, making sure they are sitting the correct way round for polarity (+ and -)
3. Make sure the volume dial is turned towards 'OFF' until it clicks
4. Plug the power adaptor into the side of the unit (DC-9V/250mA) and then into the wall. The red light (ON) will come on to show that power is getting to the unit and that it is in 'charge mode'
5. When the light turns to green the batteries are fully charged. Remove the power adaptor and the unit is ready for use
6. Fit the head-worn microphone around the back of your head so that the two loops fit comfortably over your ears and the microphone sits close to your mouth. Plug into the MIC IN socket on the unit.
7. Click the volume dial 'ON'. The red 'PW-ON' light should come on to show that the unit is now switched on
8. Speak into the microphone and adjust the volume setting as required
9. Remember to switch off the power when not in use to ensure optimum battery life

### Specifications

Frequency response	50Hz-12KHz
Max output power	6W
Output impedance	4 Ω
Operating voltage	6x 1.2V Ni-Mh rechargeable or 6x 1.5V Alkaline batteries
Operating temperature	-10-45°C
Charge time	6-8 hours
Operating time	4-6 hours continued use (new/ fully charged batteries)

### Contents of box

<ul style="list-style-type: none"> <li>• mini voice amplifier</li> <li>• adjustable waist strap</li> <li>• belt clip</li> <li>• head-worn microphone*</li> <li>• 6x rechargeable AAA batteries</li> <li>• operating instructions leaflet</li> <li>• power adaptor</li> </ul> <p>*Microphone description and specification will depend on your purchase option. Assessment kit contains: 3x Headworn Microphones, Throat Microphone and Lapel Clip Microphone.</p>
---

### Important notes

- Make sure the batteries are fully charged before their first use
- Do not leave the batteries in the unit when not in use for a prolonged period of time
- When the power light starts to dim, the battery power is low. Change or recharge the batteries as soon as possible
- For best results, make sure the source of sound is close to the microphone
- Keep the microphone away from the speaker to avoid any feedback on the amplifier
- Keep the unit upright and use the waist strap provided when in use
- The unit will operate using the mains adaptor without batteries. For your safety, we do not recommend that this method is used unless in the case of an emergency

USERMVA2EN