

Introduction

This exercise will teach you how to turn the Lightwriter on and start creating messages

functions used:



1. Press the switch to turn the Lightwriter ON
2. Enter some words. Use **Space** for gaps between words.
3. Use **<** (*Backspace*) to correct mistakes
4. Select **↶** (*Replay*) to replay your message
5. To clear the screen and the last message(s): select **⇒** then **Clear**
6. Select **⦿** to summon attention.
Select **⇒** then **⇒** then **Alarm** for a different alarm.
7. To switch the Lightwriter OFF: select **⇒** then **Off**