


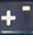



Setting up Multiple Word Predictions on an SL87

The scan screen can be set to display eight predictions instead of just one, each selected using the number keys. It cannot be used at the same time as the Extended Text mode. This can be used only in direct selection mode. This feature is ideal for people with word finding difficulties and head-pointer users in particular.



1. Press **On/C**
 2. Press **Mem** then  then  then  to enter *Advanced Setups*, and additional setups are now added to setup list.
 3. Press the **K** key repeatedly to find **Kbd screen** setup.
 4. Change the setting to **Predict** using  key.
 5. Press the **P** key repeatedly to find **Predict after** setup.
 6. Change the setting to **1, 2 or 3** using  key.
 7. Press **Off**
 8. The system is now ready for use.
-