




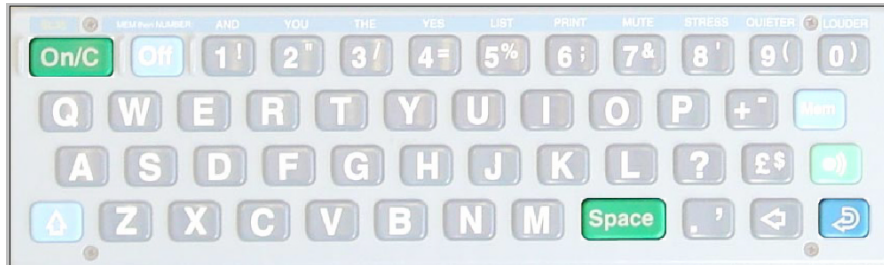


Word Prediction

This exercise will teach you how the Lightwriter's Word Prediction operates. This may help to speed up your typing. Try it and see.

Keys used:


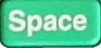


The Word Prediction can be set to predict after 1, 2 or 3 letters. This is changed in the Lightwriter's setups.

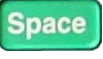1. Type in the first letter of the word
The Lightwriter predicts what you might want to say. It shows the prediction in small letters after the letter you have entered, eg **T** will predict **The**
2. If this is the word you were thinking of: press 
This will enter the whole word, followed by a space.
3. If the letter you have typed is the word you want:
press 
4. If it is not the right word, enter the next letter. Notice that the word prediction changes.
5. If this is the word you were thinking of: press 
6. If the letters you have typed is the word you want:
press 
7. Continue until the word you want is shown.
8. If the Word Prediction stops showing any more words: type the rest of the word, followed by , as normal.


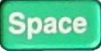
Eg: to type the word **TOMORROW**

Word Prediction after one letter

1. Type in **T** The Lightwriter predicts **The**
2. Type in **TO** The Lightwriter predicts **Too**
3. Type in **TOM** The Lightwriter predicts **TOMorrow**
4. To select **TOMORROW**: press 
5. To select **TOM**: press 


Word Prediction after 2 letters

The Lightwriter will not show any predicted words until you have typed 2 letters.

1. Type in **T**
2. Type in **TO** The Lightwriter predicts **Too**
3. Type in **TOM** The Lightwriter predicts **TOMorrow**
4. To select **TOMORROW**: press 
5. To select **TOM**: press 

Word Prediction after 3 letters

The Lightwriter will not show any predicted words until you have typed 3 letters

1. Type in **T**
2. Type in **TO**
3. Type in **TOM** The Lightwriter predicts **TOMorrow**
4. To select **TOMORROW**: press , or
To select **TOM**: press 